Manipolo 1

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Hand-piece 1

- Head A dimensions 35x25 mm
Radiofrequency at emission points Matrix for face treatments
- Head B dimensions 52x38 mm
Radiofrequency at emission points Matrix for décolleté treatments
- Matrix RF (RF Resistive)**, Matrice di punti di emissione, 400KHz
- The special geometry of the electrode and the type of emission manage to evoke a microneedling effect, very effective on the epidermis and fibroblast structures, since for its physical specifications it is able to cross easily the epidermis and the surface structures of derma.
- In fact, thanks to the fractional signal and to the energy emission matrix, it is possible to concentrate the energy at deep levels of derma by thus evoking significant and immediate effects on skin restructuring, by also stimulating an important oxygenating and regenerating action.**
- Head C dimensions 35x25 mm
IR Led + Hyperthermic RF/Athermal RF/Electrical stimulation
- Head D dimensions 35x25 mm
Red Led + Hyperthermic RF/Athermal RF/Electrical stimulation
- Head E dimensions 35x25 mm
Blue Led + Hyperthermic RF/Athermal RF/Electrical stimulation
- Head F dimensions 35x25 mm
Hyperthermic RF/Athermal RF/Electrical stimulation
- IR Photobiostimulation (850nm)**

Luminous emission able to penetrate deeply on tissues, used to generate a deep diathermic effect able to stimulate the cellular metabolism, by facilitating the cellular turn-over and also the metabolic reduction of the deep fat layer.

RED Photobiostimulation (616nm) [THIS FUNCTION CAN BE USED ONLY ABROAD ITALY]

Specific luminous emission able to interact with cellular mechanisms by increasing all activities aimed at the improvement of the skin trophism, by promoting the photostimulation of particular mitochondrial proteins.

BLUE Photobiostimulation (470nm) [THIS FUNCTION CAN BE USED ONLY ABROAD ITALY]

Specific electromagnetic emissions able to have a bactericidal action even against the organisms responsible of acne. In fact, the Blue/Violet light provokes a photooxidation reaction of the porphyrins by producing oxygen and therefore having an anti-bactericidal action.

Hyperthermic RF (Bipolar Resistive RF 400KHz)

Innovative technology able to have anti-aging effects.

The generated thermal effects give an immediate tension effect on the skin by acting on the oxygenation and stimulation of the existing collagen, but also long-term stimulation effects of the new structural proteins by the Fibroblasts.

Athermal RF (Bipolar Resistive RF 350KHz, modulating 42Hz)

Thanks to special electromagnetic emissions, this type of RF interacts directly with the skin cellular nature, by obtaining a pH balance, and also many tissue humours, by bringing back skin to its normal balance. In this treatment the radiofrequency signal is modulated, so that to have effects specifically on specific ionic channels, granting a series of reactions with anti-inflammatory effect and increase of intra/extracellular exchanges.

Electrical Stimulation (bearing frequency 100KHz, modulating 42Hz)

Specific electromagnetic emissions able to create some priority channels through the skin, by allowing the penetration on it of specific substances able to have important effects on the whole tissue section. According to the problem, they can be conveyed antioxidants, vitamins, proteins etc.

Hand-piece 2

- Head A dimensions 35x25 mm
- Head B dimensions 52x38 mm

Vacuum (max 50kPa - Ton 1 sec. T.off 1 sec.)

Derm mechanical stimulation through the use of pulsed vacuum phases, to induce the increase of microcirculation, the drainage and also a series of reactions of subcutaneous tissue re-adjustment, by remodeling the derma fibrousness. The vacuum treatment makes a massage of derma structures by improving the lymphatic circulation and making uniform the tissue consistency.

Press (max 130 kPa - Ton 1 sec. T.off 1 sec.)

Press is a type of treatment that provides for rhythmic push phases, with air, able to generate some subcutaneous mechanical vibrations. These vibrations, by interacting with the skin receptors, are able to stimulate different effects including toning, microcirculation increase and therefore anti-oxidant effect.

Hand-piece 3

Elettromiostimolazione (bearing frequency 7500Hz, modulated at 75Hz)

Type of treatment studied to stimulate the face muscular layer.

Often, together with chrono-ageing of the skin it is associated a loss of vigour of the underlying muscular layer, with consequent loss of face tone and formation of folds, wrinkles and more evident relaxations. The skin droops on the muscular layer and therefore a loss of tone of this latter will lead to a tired and saggy aspect, also by making less effective the face treatments just finalized to epidermis treatment.

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SAUNA ITALIA®

SIMMUP Face

New generation device for the treatment of face imperfections.

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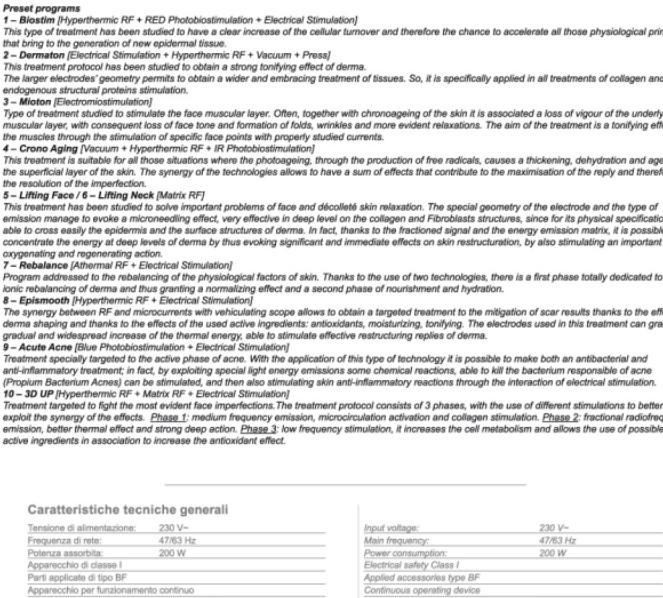
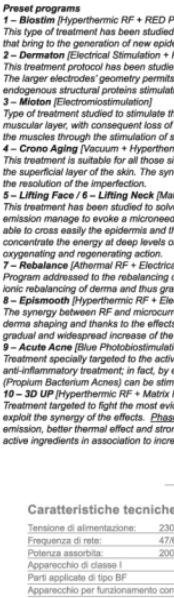
EMS

Preset programs

1 - Biostim [Hyperthermic RF + RED Photobiostimulation + Electrical Stimulation]

This type of treatment has been studied to have a clear increase of the cellular turnover and therefore the chance to accelerate all those physiological principles that bring to the generation of new epithelial tissue.

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This treatment protocol has been studied to obtain a strong tonifying effect of derma.

The larger electrode's geometry permits to obtain a wider and embracing treatment of tissues. So, it is specifically applied in all treatments of collagen and all endogenous structural proteins stimulation.

3 - Moton [Electromotostimulation]

Type of treatment studied to stimulate the face muscular layer. Often, together with chronoageing of the skin it is associated a loss of vigour of the underlying muscular layer, with consequent loss of tone and formation of wrinkles and more evident relaxations. The aim of the treatment is a tonifying effect on the muscular layer through the stimulation of a series of points with properly studied currents.

4 - Crano Aging [Vacuum + Hyperthermic RF + IR Photobiostimulation]

This treatment is suitable for all those situations where the photoaging, through the production of free radicals, causes a thickening, dehydration and ageing of the superficial layer of the skin. The synergy of the technologies allows to have a sum of effects that contribute to the maximisation of the reply and therefore the resolution of the imperfection.

5 - Lifting Face / 6 - Lifting Neck [Matrix RF]

The fractional radiofrequency has been studied to solve important problems of face and décolleté skin relaxation. The special geometry of the electrode and the type of emission manage to evoke a microscindence effect, very effective in deep level on the collagen and Fibroblasts structures, since for its physical specifications it is possible to easily cross the epidermis and the surface structures of derma. In fact, thanks to the fractioned signal and the energy emission matrix, it is possible to concentrate the energy at deep levels of derma by thus evoking significant and immediate effects on skin restructuring, by also stimulating an important oxygenating and regenerating action.

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Programme designed to take advantage of the synergies of the physiological factors of skin. Thanks to the use of two technologies, there is a first phase totally dedicated to the ionic rebalancing of derma, and thus giving a normalizing effect and a second phase of nourishment and hydration.

8 - EpisMOOTH [Hyperthermic RF + Electrical Stimulation]

The synergy between RF and microcurrents with a circulatory scope allows to obtain a targeted treatment to the mitigation of scar results thanks to the effects of derma shaping and thanks to the effects of the useful active ingredients: antioxidants, moisturizing, tonifying. The electrodes used in this treatment can grant a gradual and widespread increase of thermal energy, able to stimulate effective restructuring replies of derma.

9 - Acute Acne [Blue Photobiostimulation + Electrical Stimulation]

Treatment specifically targeted to the evolution of acne. With the application of this type of technology it is possible to make both an antibacterial and anti-inflammatory treatment; in fact, by exploiting the useful light energy emissions some chemical reactions, able to kill the bacterium responsible of acne (Propium Bacterium Acnes) can be stimulated, and then also stimulating skin anti-inflammatory reactions through the interaction of electrical stimulation.

10 - 3D UP [Hyperthermic RF + Matrix RF + Electrical Stimulation]

Treatment targeted to fight the most evident face imperfections. The treatment protocol consists of 3 phases, with the use of different stimulations to better exploit the synergy of the effects. Phase 1: medium frequency emission, microcirculation activation and collagen stimulation. Phase 2: fractional radiofrequency emission, better thermal effect and strong deep action. Phase 3: low frequency stimulation, it increases the cell metabolism and allows the use of possible active ingredients in association to increase the antioxidant effect.

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